

TESTING FOR Sexually Transmitted Infections

How to Swab Your Anus




1

Wash your hands.



2

Get into a comfortable position.

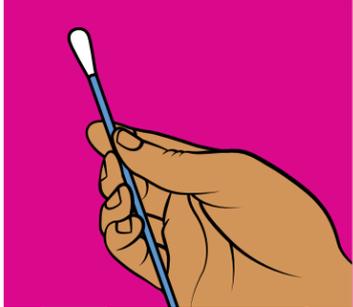


3

Hold the swab 2 inches (5 centimeters) from the tip.



4

Insert the swab 1 inch (2.5 centimeters) into your anus.

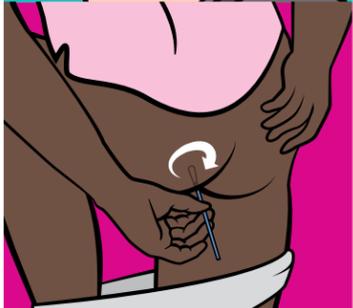

5

Rotate the swab five times in your anus and remove it.



6

Without putting down the swab, unscrew the cap.



7

Place the swab in the tube and break off the end of the swab.



8

Tightly screw the cap back on.



9

Wash your hands.

DO NOT pour out the liquid in the tube or break the foil cap.

